




Instructions For Use




1

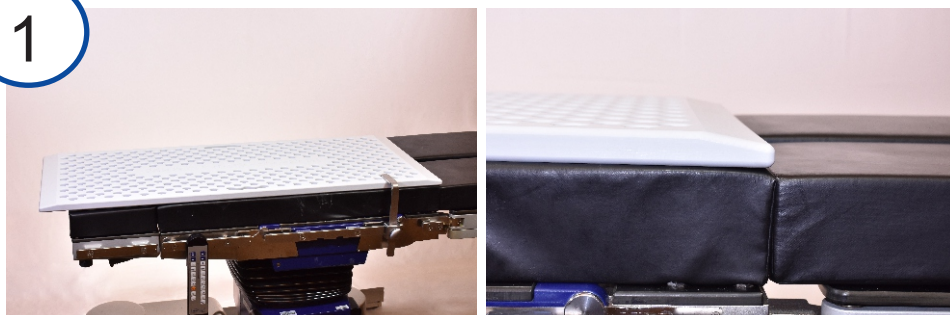

- Place pegboard on table with tapered ends up and bottom of board near the break in the bed.

2

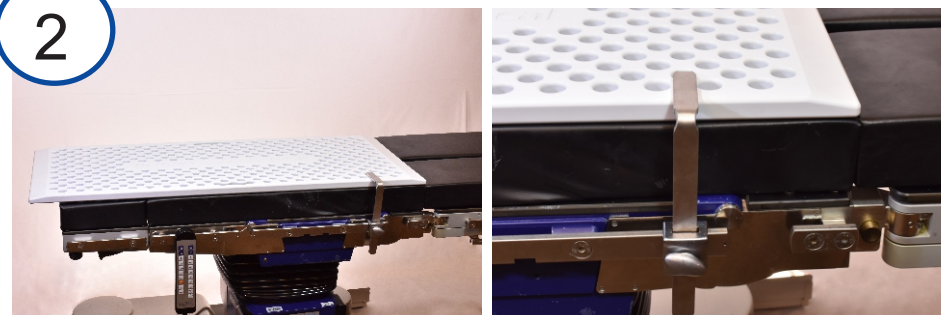


- Place mounting bracket at bottom of board near table break.

3



- Place second mounting bracket near the opposite corner of the board.

4

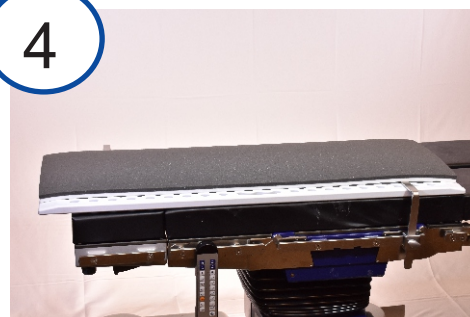


- Place foam pad flat on board per manufacturer markings.

5



- Lay patient on table as per standard procedure protocol.
- Place head and arm supports as necessary.

6

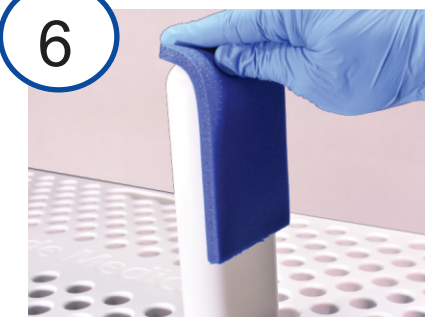

- Adhere Broadhead Peg Pads to pegs by pressing peg and pad together.

7



- Turn patient on side.
- Posterior pegs are placed: near shoulder blades, in the lumbar area, and behind non-operative leg.
- If overlay pad and peg pads don't provide ample coverage between pegs and skin, use additional padding where necessary.


8




- Anterior pegs are placed: near pubic region and near sternum.
- Use additional padding where necessary.




The Solorail Standard Kit Includes:

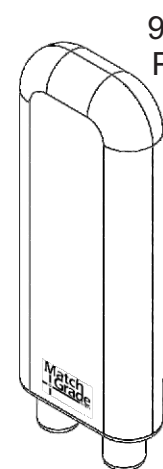


9" Double Wide
Pegs - Set of 2
Included
(HP 9020)

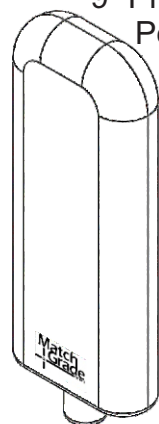


9" Pivot Double Wide
Pegs - Set of 2
Included
(HP9030)

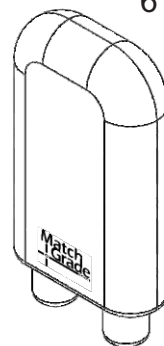


6" Double Wide
Pegs - 1
Included
(HP602)

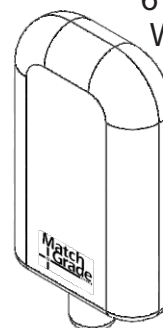


6" Pivot Double
Wide Pegs - 1
Included
(HP603)

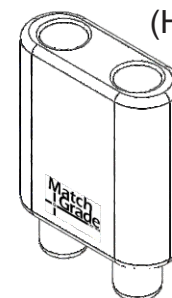

4" Double Wide
Peg Extensions -
Set of 2 Included
(HP4020)

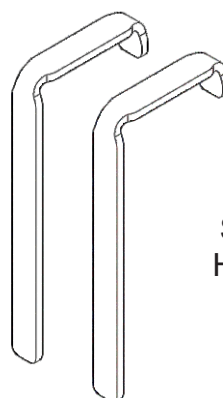


Surgical Positioning
Hold-Down Clamps -
Set of 2 Included
(HPC4)

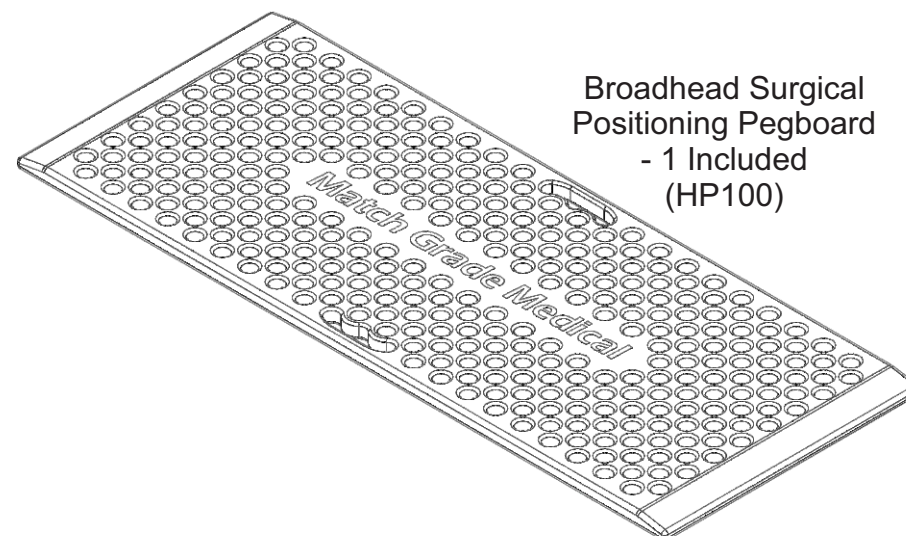

Broadhead Surgical
Positioning Pegboard
- 1 Included
(HP100)

Also Available:

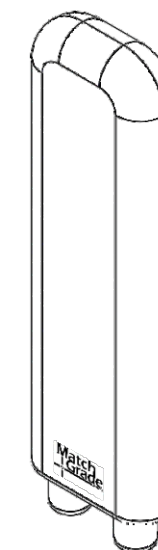

12" Double Wide
Pegs - Set of 2
(HP1220)

(Not Included in
Standard Kit)

ACCEPTANCE OF TERMS AND CONDITIONS OF USE. By choosing to use the Broadhead Peg Board System, the user agrees to be bound by these Terms and Conditions of Use as supplied with the Broadhead Peg Board when delivered. A copy of the Instructions For Use along with the Terms and Conditions of Use is available from Match Grade Medial.

NOT REQUIRED FOR USE. The Broadhead Peg Board System is not necessary for the successful completion of surgery. The choice to use the Broadhead Peg Board is at the full discretion of the surgeon to aid in the convenience of repeatable and stable positioning of patient's lower extremities. Match Grade Medical makes no claims that the use of the Broadhead Peg Board will in any way improve the outcome of the surgery, nor will the decision to discontinue use of the Broadhead Peg Board System detract from the outcome of the surgery.

LIMITATION OF WARRANTY. THERE ARE NO WARRANTIES, EXPRESS OR IMPLIED, THAT EXTEND BEYOND THE DESCRIPTION OF THE PRODUCTS, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE. However, Seller warrants that the Products delivered shall conform to the grade and quantity specified in applicable documents published by Match Grade Medical.

LIMITATION OF LIABILITY. SELLER'S LIABILITY SHALL BE LIMITED TO THE COST OF REPLACING DULY REJECTED PRODUCTS TO THE POINT OF DELIVERY AS SPECIFIED IN THE ORDER ACKNOWLEDGMENT WITHIN A REASONABLE PERIOD OF TIME FOLLOWING THE ISSUE OF RETURN MATERIAL AUTHORIZATION BY MATCH GRADE MEDICAL. UNDER NO CIRCUMSTANCES SHALL SELLER BE LIABLE TO BUYER FOR CONSEQUENTIAL OR INCIDENTAL DAMAGES, OF ANY NATURE WHATSOEVER, ARISING FROM NONCONFORMITY OF PRODUCTS OR DELAY IN SHIPMENT, OR ANY OTHER BREACH BY SELLER.

EXCLUSIVE REMEDY. Buyer's sole and exclusive remedy for breach of any warranty hereunder is the repair or replacement of any defective goods or services by Match Grade Medical at the discretion of Match Grade Medical. Under no circumstances shall the Match Grade Medical be liable for any value in excess of the purchase price of the goods or service.

GENERAL. The rights and obligations of the parties under the Terms and Conditions of Use shall be governed by the laws of the State of Wisconsin, U.S.A. in effect as of the date of the initial delivery, including without limitation the provisions of the Wisconsin Uniform Commercial Code, but without regard to conflicts of law principles. If any provision of the Terms and Conditions of Use is determined by any court or arbitrator to be unenforceable, the provision shall be deleted, and the balance of the Terms and Conditions of Use shall be binding upon the parties.